# M@bility

NETWORKS

Sollevatori dis edie a rotelle

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***i-class***

Wheelchair lift

Original instruction translation

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English text of this document is the translation of the original manufacturer instructions. If further assistance is required, please refer to the original document or forward a request to **Mobility Networks Holdings Ltd.**

This handbook cannot be reproduce, stored by a filing system or transmitted to third parties, in whole or in part, in whatsoever form or medium, whitout **Mobility Networks Holdings Ltd.** prior written permission.

**Mobility Networks Holdings Ltd.** reserve the right to carry out products changes and improvements in order to enhance their quality, at any time and without notice, even il such modifications will not he reported in this publication.

**The Testing and Inspection operations of this product must be performed by authorized Mobility Networks Holdings Ltd. centres.**

**Keep this manual in the vehicle for future reference.**

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## 1. LOG VERIFICATION FOR THE FIRST COMMISSIONING

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*Compilare i campi come descritto al Cap. 4 del manuale di Installazione e Service*  
 Compile the fields as described to the *Chapter 4 of the Fitting and Service handbook.*

|                  |                     |                      |                      |
|------------------|---------------------|----------------------|----------------------|
| <b>P</b> = _____ | <b>A</b><br>= _____ | <b>d1</b><br>= _____ | <b>G1</b><br>= _____ |
| <b>S</b> = _____ | <b>B/B*</b> = _____ | <b>d2</b><br>= _____ | <b>G2</b><br>= _____ |

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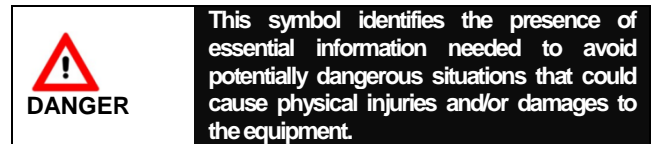
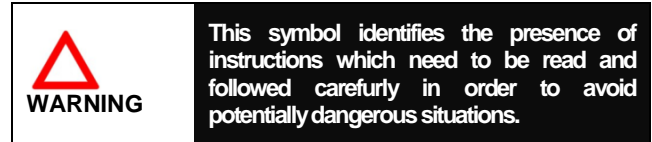
## **SAFETY RULES AND SYMBOLS**

Some precaution are listed in this manual in order to identify potentially dangerous situation in advance and the relevant instructions to avoid them, identified by the following symbols:

To ensure smooth and safe operation, it is necessary to know the exact procedure for the installation and service of the wheelchair lift.

### **WARNING**

Carefully follow the instructions for the installation and service of the wheelchair lift within this manual. If the instructions within this manual have not been fully understood or further information is required, please contact Mobility Networks Holdings Ltd. immediately. These indications non-fulfilment can cause serious damages, to the equipment and to the vehicles as well as serious injuries to people.

### **DANGER**

Should it be necessary to raise the vehicle being equipped, please check that the characteristics of the lifting device or of the mobile bridge are compatible with the volume and weight of the actual vehicle otherwise there is a risk of serious damage to equipment and vehicles as well as serious injuries to people.

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**INDEX**

|   |    |
|---|----|
| 1. LOG VERIFICATION FOR THE FIRST COMMISSIONING ..... | 3  |
| 2. Declaration of conformity.....                     | 6  |
| 3. PREFACE.....                                       | 9  |
| 3.1. Classification and general considerations .....  | 9  |
| 3.2. Documentation .....                              | 9  |
| 3.3. Warranty.....                                    | 9  |
| 3.4. Technical Description .....                      | 11 |
| 3.5. Technical specifications.....                    | 12 |
| 3.6. Models and Installations .....                   | 12 |
| 3.7. Overall dimensions .....                         | 13 |
| 4. STANDARDS AND SAFETY DEVICES                       | 15 |
| 4.1 Safety Regulations.....                           | 15 |
| 4.2 Safety Devices.....                               | 19 |
| 4.3 Identification Plates and Stickers .....          | 20 |
| 5. CONTROLS.....                                      | 22 |
| 6. USE AND OPERATION .....                            | 24 |
| 6.1. Pre-operation checks.....                        | 24 |
| 6.2. Lift switching on .....                          | 25 |
| 6.3. Platform deploying .....                         | 26 |
| 6.4. Lowering.....                                    | 27 |
| 6.5. Lifting.....                                     | 28 |
| 6.6. Stowing.....                                     | 29 |
| 6.7. Disabling the system.....                        | 30 |
| 6.8. Control before vehicle movements.....            | 30 |
| 6.9. Expected conditions of special use .....         | 31 |
| 6.10. Emergency Operation .....                       | 31 |
| 7. MAINTENANCE .....                                  | 33 |
| 7.1 Periodic visual checks .....                      | 33 |
| 7.2 Clening of the loading platform.....              | 35 |
| 7.3 Routine inspections.....                          | 36 |
| 7.4 Special Inspection .....                          | 47 |

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### 3. PREFACE

#### 3.1. Classification and general considerations

The wheelchair lift “i-class”, manufactured by **Mobility Networks Holdings Ltd.**, which is the object of this instruction manual, is in compliance with the safety requirements of 2006/42/CE directive. **i-class** wheelchair lift, as per the aforementioned directive, must be considered a ‘machine’ and therefore a declaration of conformity has been released, according to annex II of 2006/42/CE directive.

All the necessary instructions for the correct installation of the wheelchair lift are detailed for the installer together with a list of instructions for servicing procedures and for extraordinary maintenance.

 **WARNING** All installation and service procedures must be carried out in compliance with current health and safety laws.

#### 3.2. Documentation

The user should receive with the “i-class” wheelchair lift:

- the original copy of the manufacturer's declaration of conformity;
- the installation conformity declaration both filled and signed by the installer;
- use and maintenance manual filled by the installer.

**Mobility Networks Holdings Ltd.** is at your complete disposal for further clarification and instructions.

#### 3.3. Warranty

**Mobility Networks Holdings Ltd.** assures the wheelchair lift for 24 months from the date of delivery. The warranty covers for defects concerning material quality and product manufacturing.

The warranty does not cover consumables and defects or failures as a result of installation procedure incorrectly carried out or not in compliance with the manufacturer's instructions.

The warranty becomes void in case of impacts caused by accidents and/or tampering carried out by personnel not authorized by **Mobility Networks Holdings Ltd.**

**Mobility Networks Holdings Ltd.** refuses all responsibility for damages caused by:

- improper use of the electro-hydraulic lift;
- lift maximum load capability (425Kg);
- a failure in carrying out “Use and Maintenance” manual instructions;
- a failure in carrying out maintenance operation as detailed in the “Use and Maintenance” manual;

## Mobility Networks

12 Estuary View Business Park, Whitstable, Kent, CT5 3SE, UK

Tel: +44 (0)1227 505022 Fax: +44 (0)1227 505023 [www.mobilitynetworksgroup.com](http://www.mobilitynetworksgroup.com)

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- interventions or modifications to the lift without **Mobility Networks Holdings Ltd.** authorization;
- lift safety devices removed or tampering;
- wrong manouvres when moving the hydraulic lift;
- presence of objects or people in lift working area.

 **WARNING** Manufacture's regular inspections dates non-fulfilment will automatically make void the warranty.

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### 3.4. Technical Description

**i-class** model wheelchair lift (**Fig. 1**) is installed on the deck of vehicles used for transporting disabled persons in wheelchairs, allowing them to get in/off the vehicle.

Lift consists of a base fixed to vehicle loading deck a pair of lifting articulation, installed on the sides of the base, and a loading platform, hinged between the articulations.

Deploying/stowing and lifting/lowering movements of the lift are made by means of a parallelogram leverage mechanism driven by a pair of hydraulic cylinders (one for each articulation). The machine is equipped with a hydraulic control unit and an electronic control box which, by means of a remote control, plays the various functional movements; the entire system is electrically powered by the batteries of the vehicle on which it is installed.

Left side

Right side

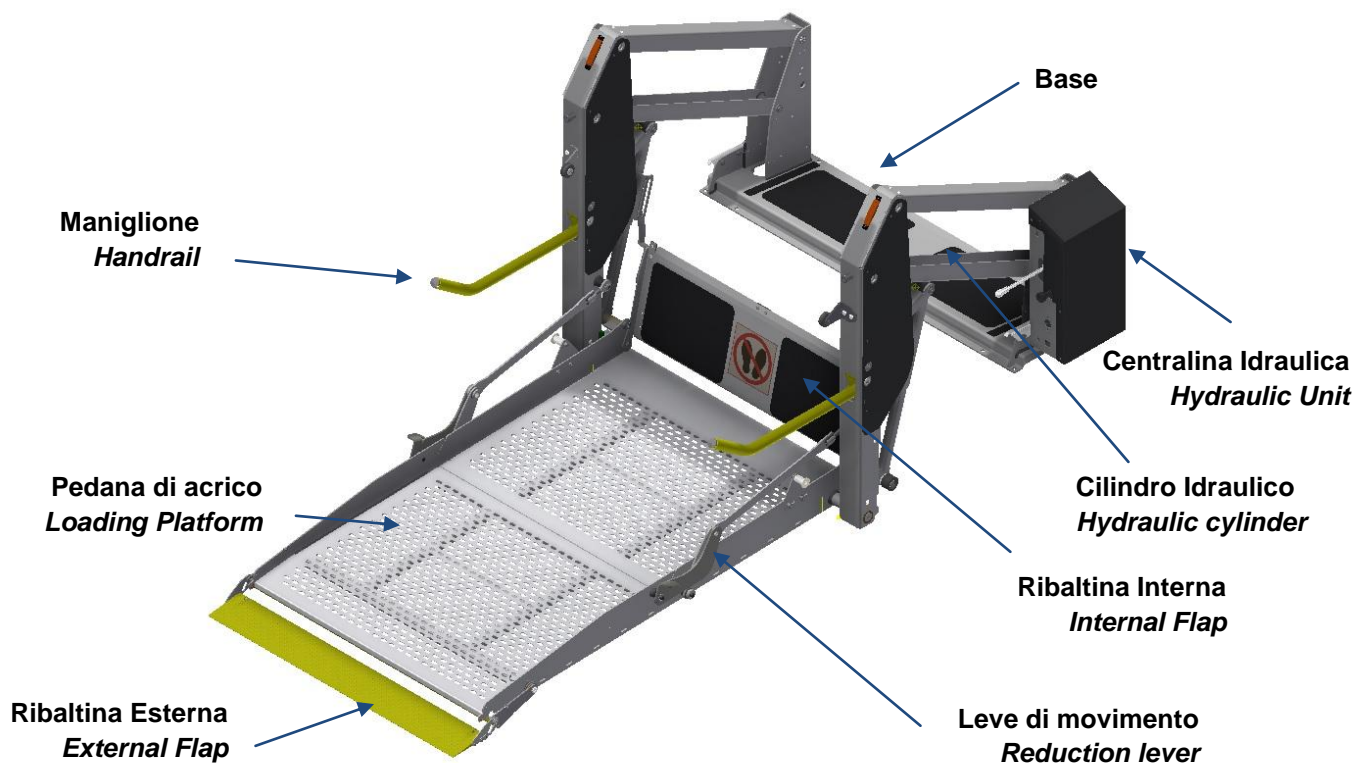
Fig.1

**Mobility Networks**

12 Estuary View Business Park, Whitstable, Kent, CT5 3SE, UK

Tel: +44 (0)1227 505022 Fax: +44 (0)1227 505023 [www.mobilitynetworksgroup.com](http://www.mobilitynetworksgroup.com)

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### 3.6. Models and Installations

The wheelchair lift "i-class" is available in three models, which differ from one another in type of loading platform:

- **2BB EVO FP**, with a horizontally folded one platform is deployed automatically during the opening phase and folds automatically during the stowing one, in order to reduce the overall height of the lift at rest, thereby improving the visibility of the driver of the vehicle.
- **2BB EVO P**, with a whole loading platform: during stowing phase platform simply rotates from loading position to vertical one and backwards during deploying phase.
- **2BB EVO SP**, with a loading platform split longitudinally: the two parts of the platforms rotate to form a single loading platform during the deploying phase and they are separated automatically during the stowing one, creating a space that can be used as an emergency exit from the vehicle.

It is possible to prepare the wheelchair lift with following options:

- handrails standar or UK,
- power supply voltage 12V or 24V,
- electro hydraulic power unit right or left.

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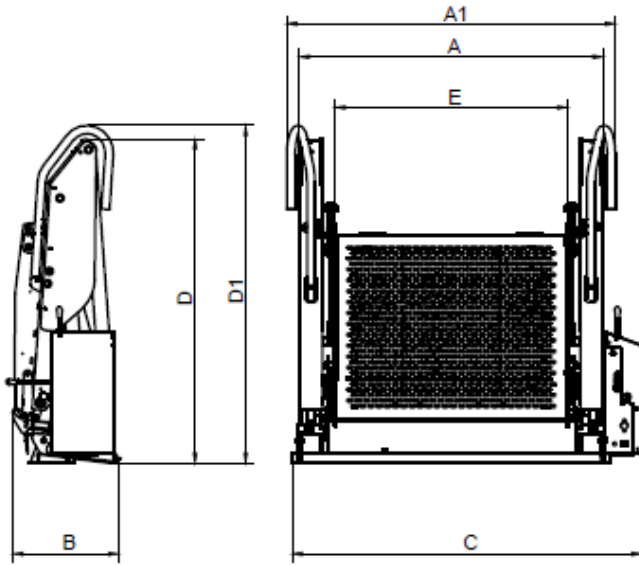
“i-class”:


### 3.7. Overall dimensions

Find following main dimensions of the versions and set-ups available of the wheelchair lift “i-class”:


**2BB EVO FP**

**2BB EVO P**

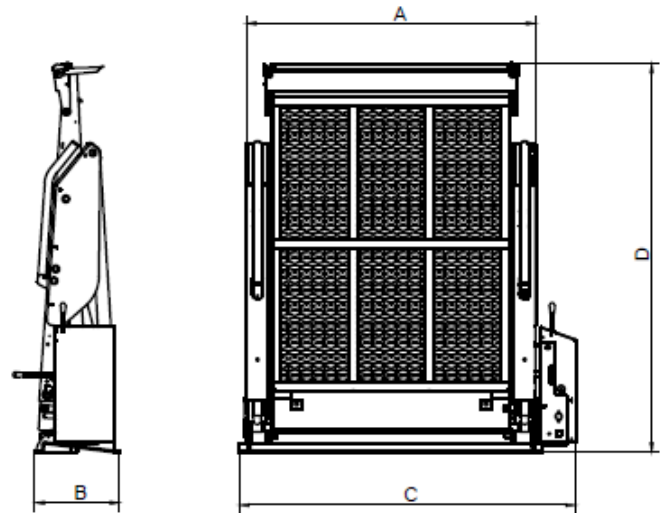**2BB EVO SP**

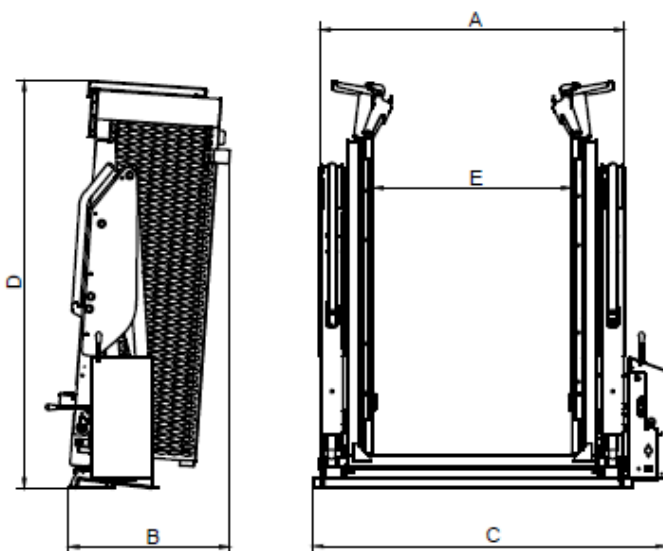
**Mobility Networks**

12 Estuary View Business Park, Whitstable, Kent, CT5 3SE, UK

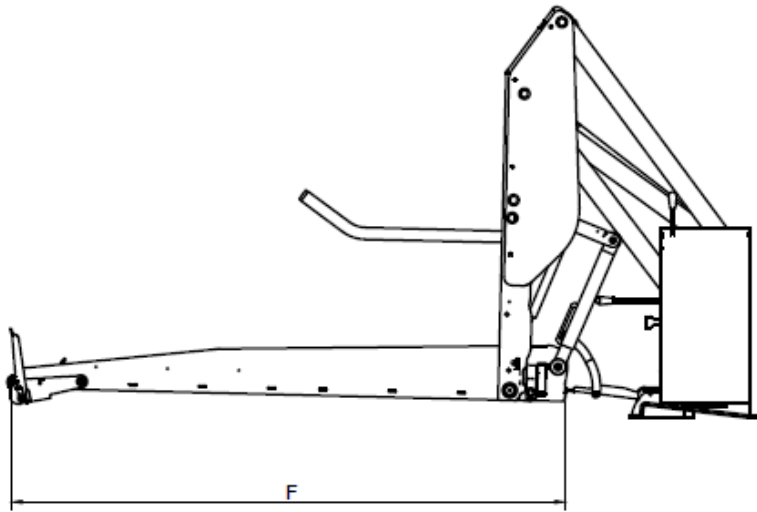Tel: +44 (0)1227 505022 Fax: +44 (0)1227 505023 [www.mobilitynetworksgroup.com](http://www.mobilitynetworksgroup.com)

| Product number                   | A           | A1(**)      | B          | C           | D           | D1(***)     | E          | F           |
|----------------------------------|-------------|-------------|------------|-------------|-------------|-------------|------------|-------------|
| <b>2BB - EVO FP84138</b>         | <b>1106</b> | <b>1196</b> | <b>385</b> | <b>1275</b> | <b>1181</b> | <b>1230</b> | <b>840</b> | <b>1380</b> |
| <b>2BB - EVO FP80150</b>         | <b>1058</b> | <b>1148</b> | <b>385</b> | <b>1195</b> | <b>1191</b> | <b>1240</b> | <b>800</b> | <b>1500</b> |
| <i>2BB-EVO FP74138 (*)</i>       | <i>1006</i> | <i>1096</i> | <i>385</i> | <i>1175</i> | <i>1181</i> | <i>1230</i> | <i>740</i> | <i>1380</i> |
| <i>2BB-EVO FP70115 (*)</i>       | <i>966</i>  | —           | <i>355</i> | <i>1115</i> | <i>1074</i> | —           | <i>700</i> | <i>1150</i> |
| <b>2BB - EVO P90138</b>          | <b>1106</b> | <b>1196</b> | <b>325</b> | <b>1285</b> | <b>1486</b> | —           | <b>900</b> | <b>1380</b> |
| <i>2BB-EVO P80130 (*)</i>        | <i>1006</i> | <i>1096</i> | <i>325</i> | <i>1185</i> | <i>1406</i> | —           | <i>800</i> | <i>1300</i> |
| <b>2BB - EVO SP90138</b>         | <b>1106</b> | <b>1196</b> | <b>585</b> | <b>1285</b> | <b>1488</b> | —           | <b>900</b> | <b>1380</b> |
| <i>2BB - EVO SP80130 (*)</i>     | <i>1006</i> | <i>1096</i> | <i>585</i> | <i>1185</i> | <i>1408</i> | —           | <i>800</i> | <i>1300</i> |
| <i>2BB - EVO SP76100 (*)</i>     | <i>966</i>  | —           | <i>430</i> | <i>1115</i> | <i>1120</i> | —           | <i>760</i> | <i>1000</i> |
| (*) = Projects to do             |             |             |            |             |             |             |            |             |
| (**) = Width with handrails UK   |             |             |            |             |             |             |            |             |
| (***) = Height with handrails UK |             |             |            |             |             |             |            |             |

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*In this Table have the realized models, Mobiliti Networks reserves the right to update the models.*

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#### 4. STANDARDS AND SAFETY DEVICES

##### 4.1 Safety Regulations

Below are a set of rules which, will ensure that no harm is done to people, animals or things whilst using the wheelchair lift:

- The lift covered by this manual, should only be used for the lifting / lowering of disabled people in wheelchairs onto / from the vehicle where it is installed; any other use is a misuse and could cause damage to people, animals and things;
- the use of the lift is permitted exclusively to authorized personnel who are perfectly familiar with the location and the function of all the controls, lights, indicators and safety standards;
- do not use the lift if you are not sure you have understood all the manoeuvring procedures;
- failure to observe the safety rules and the instructions in this manual may result in accidents to the passenger on the wheelchair, to the operator, to the vehicle and to third parties;
- before operating the lift, carefully read all the labels and the safety stickers applied to it;
- do not wear dangling or ragged clothing (ties, scarves, loose shirts) and jewellery (watches, necklaces, bracelets) that could get caught in the moving or stationary parts of the machine;
- do not make any changes which may alter the structure of the lift/hoist that could compromise the operational security. If necessary, contact the authorized **Mobility Networks Holdings Ltd.** centre first for an optimal resolution of the problem;
- in case of tampering or unauthorized modifications, **Mobility Networks Holdings Ltd.** assumes no liability and the warranty and CE certification is deemed as lapsed;
- do not tamper with the calibration and safety devices of the lift / hoist;
- the use of the lift is forbidden:
  - in areas at risk of fire,
  - in the presence of corrosives, explosives or dust harmful to the health of the operators;
  - inside magnetic fields;
- the lift / hoist should be installed on the vehicles approved for use on public roads. Therefore, before entering into circulation, make sure that the vehicle is in compliance with the existing legislation;
- before loading the lift / hoist, make sure that the weight of the disabled passengers in addition to the wheelchair and possibly the operator, does not exceed the maximum permitted load (360kg);

##### **PRIOR TO OPERATIONS**

- always make sure that the work area is adequately illuminated, otherwise arrange for its proper illumination;

**Mobility Networks**

12 Estuary View Business Park, Whitstable, Kent, CT5 3SE, UK

Tel: +44 (0)1227 505022 Fax: +44 (0)1227 505023 [www.mobilitynetworksgroup.com](http://www.mobilitynetworksgroup.com)

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- make sure you have complete visual control of the radius of action of the lift / hoist and the whole area of work;
- verify the absence of dangers near the work area;
- ensure that all the inspections are carried out according to the predetermined timing established by the manufacturer;
- every day, before each work shift, carry out a visual check of the machine as described in Chapter 7 of this manual;
- do not use the lift in case of any damages; if necessary, contact an authorized **Mobility Networks Holdings Ltd.** service centre;
- in case of loss or deterioration of the safety tags and stickers, ensure their timely restoration;
- check that there is no leakage of fluid in the hydraulic system; otherwise, do not use the lift absolutely and request the intervention of specialized personnel for its repair;
- keep the surface of the platform clean to preserve the non-skid characteristics as described in Chapter 7 of this manual;
- when manoeuvring whilst standing on the loading platform use shoes with non-slip soles;
- always keep the loading platform free of objects so as to minimize the possibility of tripping and falling;
- carry out the lifting and lowering operations without load in order to verify that the desired action corresponds to each command; in case you experience a malfunction of the lift, suspend work, disable the lift, affix a clearly visible sign saying "OUT OF ORDER", and request the assistance of a **Mobility Networks Holdings Ltd.** specialized technician;

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**Do not use the lift in breakdown.**

**WARNING**

The lift is to be used at an environmental temperature of MIN -15°C and MAX 48°C in order to prevent malfunctions and damage to the system; in cold weather (up to max -40°C), operate the hydraulic system without load for a few minutes, so as to circulate and heat the oil.

**WARNING**

Park outside the traffic flow on a flat and level surface and make sure there is enough space for manoeuvring the lift.

**Mobility Networks**

12 Estuary View Business Park, Whitstable, Kent, CT5 3SE, UK

Tel: +44 (0)1227 505022 Fax: +44 (0)1227 505023 [www.mobilitynetworksgroup.com](http://www.mobilitynetworksgroup.com)

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 **DANGER**

Do not try to locate the oil leaks with your hands or other body parts: a leakage, even if very small, can pierce the skin causing painful sores and infections that require prompt medical intervention.

**DURING THE OPERATION**

- verify that the radius of action of the lift is always free of obstructions;
- do not stop and do not let people stop in the radius of action of the lift;
- the operator who operates the lift must always keep a safe distance and stay in a position that allows him to have control over the entire radius of action of the lift;
- all the manoeuvres should be done with maximum caution, keeping the head, body and limbs away from the areas of shearing, crushing, entrapping and impact;
- check that, during the movements, the platform does not interfere with any part of the vehicle, also make sure that the connecting cables of the wired remote control do not get caught in the corners of the vehicle and / or parts of the lift;
- when operating the lift do not place any objects on the push button panel and other commands;
- it is forbidden to insert objects between the moving parts of the lift;
- it is forbidden to throw objects to and from the lift;
- do not lift or to low from the loading platform of the lift when it is moving;
- the brakes should be compulsorily applied to the wheelchair before starting the manoeuvre of lifting / lowering;
- do not move the lift in operational sequences of command in contrast with those shown in *Chapter 6* of this manual;

 **DANGER**

Do not start the vehicle engine in enclosed areas, unless they are equipped with an effective ventilation system and a system for disposal of exhaust gases.

 **WARNING**

The lift has been designed exclusively for lifting and lowering disabled persons in wheelchairs with a single operator in charge to / from the floor of the car to which it is fixed; the use of the lift for loading /unloading other persons and/or objects is therefore forbidden.

 **DANGER**

The operator is required to stop the machine if the people in the radius of action of the same, do not go away despite his requests.

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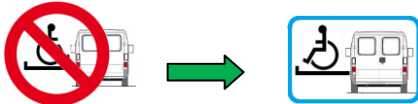
**! DANGER**

It is prohibited to exceed the nominal load; also ensure that the disabled person and any operator are always in perfect balance on the platform.

**! DANGER**

The operator must keep people who are under the dock leveler before it is lowered to prevent injury to themselves

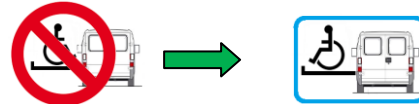
**! DANGER**

Stop any manoeuvre if there is a risk of the disabled person and/or operator falling down.

**! WARNING**

The disabled person must always be positioned with his back to the vehicle.

**! WARNING**

By withdrawing the load, the loading capacity halves at a distance.

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**AT THE END OF OPERATION**

- before closing the lift, make sure that no object was forgotten on the loading platform;
- disable the system and then disconnect and store the remote control inside the passenger compartment of the vehicle.

**! DANGER**

Before moving the vehicle check that the lift is securely closed and the indicator light in the cabin is off.

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#### 4.2 Safety Devices

The wheelchairs lift “i-class” they are equipped with devices of protection and safety:

- **Safety handrails:**  
ensure the function of "firm hold" during the lifting / lowering phase.
- **Internal flap:**  
to let wheelchair get in / off the platform; it is mounted in order to stay vertical when the platform is not at vehicle deck level in order to avoid crushing and shearing risk.
- **External flap:**  
to let wheelchair get in / off the platform; it is mounted in order to stay vertical when the platform is not at ground level in order to avoid the risk of wheelchair falling from the platform of the wheelchair.
- **Protection devices (fitted on moving parts):**  
they are carter with the function of protection from the moving parts.
- **Protection against overturning:**  
stops platform lowering in case of an obstacle which induces overturning.
- **Safety pressure switch:**  
stops stowing when platform is still loaded.
- **Protection against overloading:**  
blocks lifting in case there is a load greater than 125% of the nominal loading capability of the lift (425 Kg).
- **Protection against hydraulic system leakage:**  
avoid platform descent speed from exceeding operating one by 50%.
- **Safety hook:**  
avoid the unintentional opening of the platform in case of low pressure in the hydraulic system when the lift is closed.
- **Overall dimensions signalling:**  
intermittent led lights (optional) are provided, screwed to the top front portion of both the lifting articulation.

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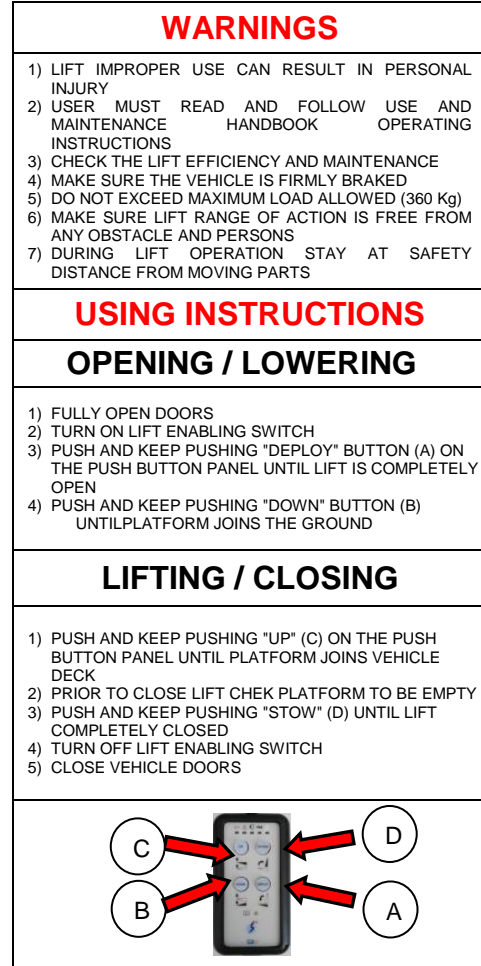
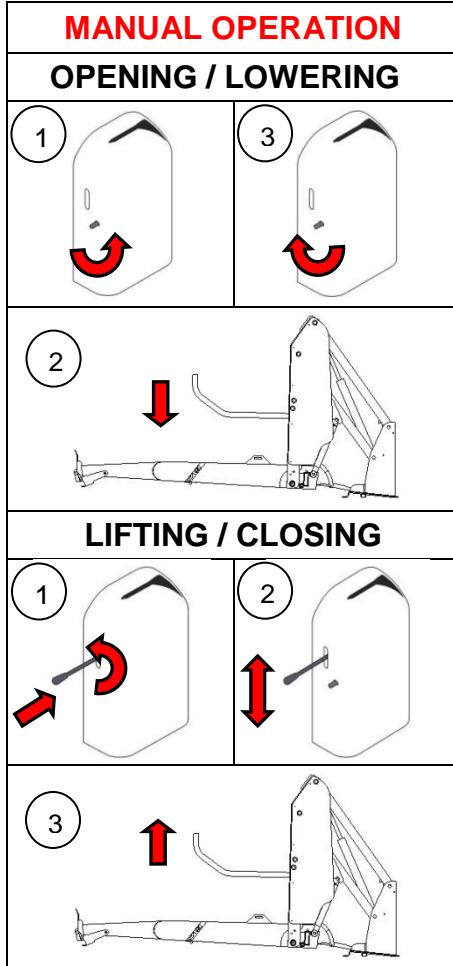
### 4.3 Identification Plates and Stickers

An identification tag and safety stickers have been applied to the lift, as follows:

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*Identification plate riveted on internal side of lift base.*

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*Do not stop lift in motion, safety sticker on the front side of lifting arms.*

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*Safety sticker on the internal flap.*

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
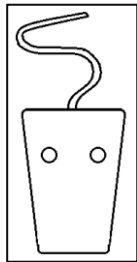
*Identification sticker of the connector of the wired remote control panel placed on the front of the control unit box.*

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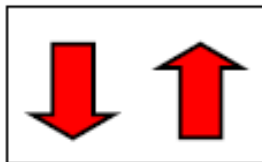*Identification sticker of the joystick command (if present) for the down/up maneuvers placed in correspondence to the related command.*

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
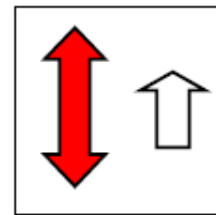
*Identification sticker of the manual pump placed on the control unit box corresponding with the slot for the hand pump lever.*

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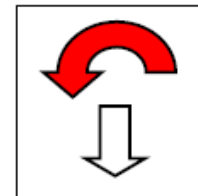*Identification sticker of the manual valve placed on the control unit corresponding to the return manual valve for descent.*

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**⚠ WARNING** In the case of loss or excessive wear, identification tags and safety stickers should be rapidly replaced. Please contact an authorized Mobility Networks Holdings Ltd. centre for details.

**Mobility Networks**

12 Estuary View Business Park, Whitstable, Kent, CT5 3SE, UK

Tel: +44 (0)1227 505022 Fax: +44 (0)1227 505023 [www.mobilitynetworksgroup.com](http://www.mobilitynetworksgroup.com)

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The remote control (**Fig. 2**) allows the operator to maneuver the lift at a safe distance from above the loading platform.

To activated the remote control connect to the connected to the CPC socket fixed on hydraulic unit cover.

To activate the wired remote control panel it is necessary to turn on the enabling system switch. If present the communication from lift at the vehicle, the enablement of the system is displayed by passenger compartment warning light switching on.

#### WARNING

It is important that the user of the lift has full vision lift warning area.

Fig. 3

The joystick control (optional) placed on the end of one of the two handrails (**Fig.4**) can be used in place of the wired remote control for the lifting/lowering manoeuvres. Push the lever upward for lifting and down ward for lowering, joystick lever is with auto return to the neutral position.

To activate the joystick command it is necessary to turn on enabling switch. If present the communication from lift at the vehicle, the enablement of the system is displayed by passenger compartment warning light switching ON.

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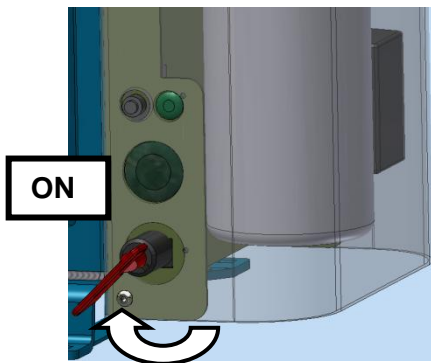
Fig. 4

## 5.2 Emergency manual controls

Emergency manual controls (see Fig. 5) are:

- Manual pump, to lift platform; it is plaiyed by means of the manual pump handle (in the scope of the delivery) wich must be inserted in the block of hydraulic unit (6);
- Manual valve to lower platform; adjustable by loosening the manual valve knob slightly of  $\frac{1}{4}$  counterclockwise (4) by means of lever A.

## 5.3 Hydraulic unit

The hydraulic unit of the lift can be delivered in two versions: manual controls on the left side (left version) or manual controls on the right side (right version); both versions are suitable only for vertical installation.

- 1) Electro-pump electric engine
- 2) Seat of manual pump
- 3) Adjustable throttle
- 4) Knob lowering electro valve
- 5) Contactor
- 6) Manual pump
- 7) Cap
- 8) Oil tank

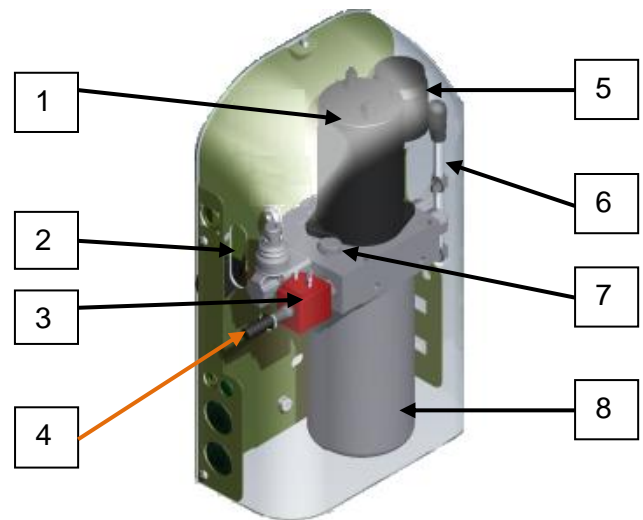
Fig. 5

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## 6. USE AND OPERATION

### DANGER

Before starting using the lift, you must properly brake the vehicle by activating the parking brake and, if necessary, you must use wheel clamp wedges.

### WARNING

Ensure that the vehicle is parked on a flat surface, horizontal and free of clutter.

#### 6.1. Pre-operation checks

Before using the wheelchair lift it is mandatory to perform the following steps:

- ensure that there are no missing or damaged parts of the machine and structural failure;
- check that the safety labels and stickers are visible and legible;
- check vehicle batteries charge level;
- check hydraulic oil level contained in the tank and, if necessary, fill it up with oil having the same features. These operations must be carried out with platform on the ground;
- always keep the machine moving elements clean from dirt;
- check that all pins are correctly housed in their seats, and that they are not lacking of their respective locking stops;
- check the integrity of the spiral cable of the wired remote control and the electrical connectors;
- check for leaks of hydraulic oil;
- check the fasteners for all the pins of the lifting arms;
- ensure that there are no loose screws.

**In case of any kind of failure, do not use the lift and contact an authorized Mobility Networks Holdings Ltd. centre.**

### DANGER

Do not start the vehicle engine up in closed areas if they are not provided with efficient ventilation and exhaust gas disposal systems.

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### WARNING

Make sure that the relative movement of the platform corresponds to each command without jamming and unusual noises.

### DANGER

If there are movements of the platform that are different from those provided for that particular command, do not use the lift and contact an authorized Mobility Networks Holdings Ltd. centre.

#### 6.2. Lift switching on

To carry out the operation, comply with the following instructions:

- Plug the battery isolator spanner (**Fig. 6**) in the switch and rotate it clockwise;
- Connect remote control to the CPC socket fitted above the battery isolator (**Fig. 6**).
- The enablement of the system is displayed by passenger compartment warning light switching ON.

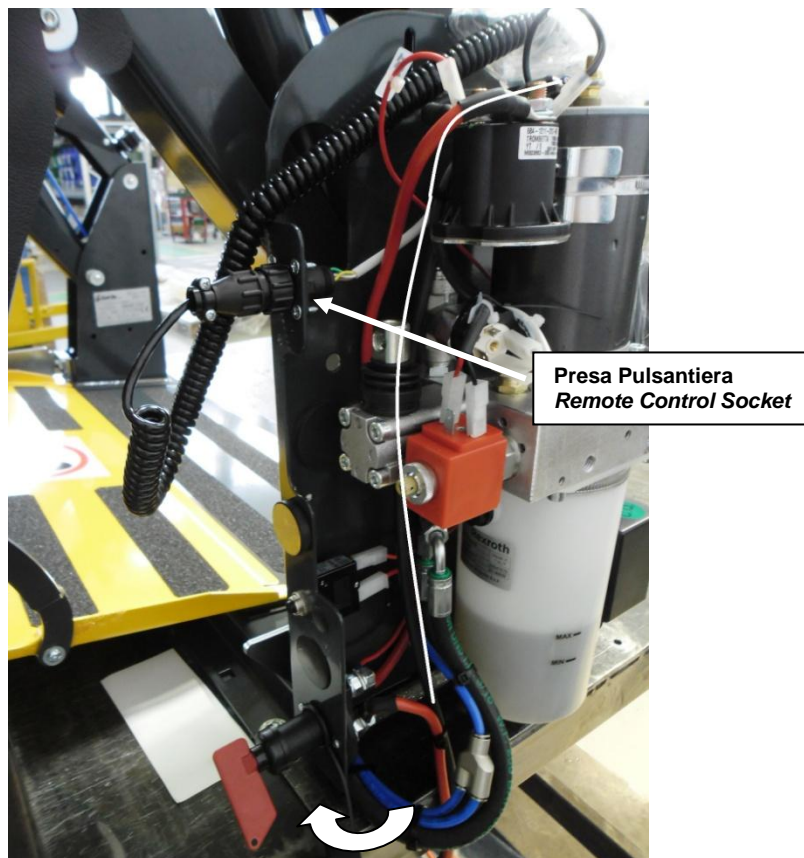
Fig.6

#### Mobility Networks

12 Estuary View Business Park, Whitstable, Kent, CT5 3SE, UK  
 Tel: +44 (0)1227 505022 Fax: +44 (0)1227 505023 www.mobilitynetworksgroup.com

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### 6.3. Platform deploying

Opening is a rotational movement that leads the loading platform from the vertical (close) position to the horizontal loading position on the floor of the vehicle.

Follow these steps for the deploying manoeuvre:

- connect the wired remote control panel to the relative connector, **Chapter 5**,
- push and keep pushing button **1** in **Fig.2 Chapter 5** for **deploy (Fig.8)** until platform stops in the horizontal loading position on the floor of the vehicle (**Fig.9**);
- release button.

Only at this point platform is enabled to be loaded.

**NOTE** Same task can be plaied by means of wireless remote control (optional).

**NOTE** When opening/closing door automatic system 8optional) is foreseen by pushing and keeping pushing button **1** in **Fig.2 Chapter 5** system first opens vehicle doors and then it deploys the loading platform (only when doors are completely open).

**⚠** If the platform during the deploying manoeuvre is locked, make sure if the safety hook (**Fig.7**) failed to act; otherwise it is necessary to return the platform to the closed position by pressing "stow" **4** button (see **Fig.2 Chapter 5** and then repeat the opening manoeuvre described in *Chapter 5*

**⚠** If there are movements of the platform that are different from those provided for that particular command, do not use the lift and contact an authorized CAR OIL System

**⚠** Make sure that the relative movement of the platform corresponds to each command without jamming and unusual noises.

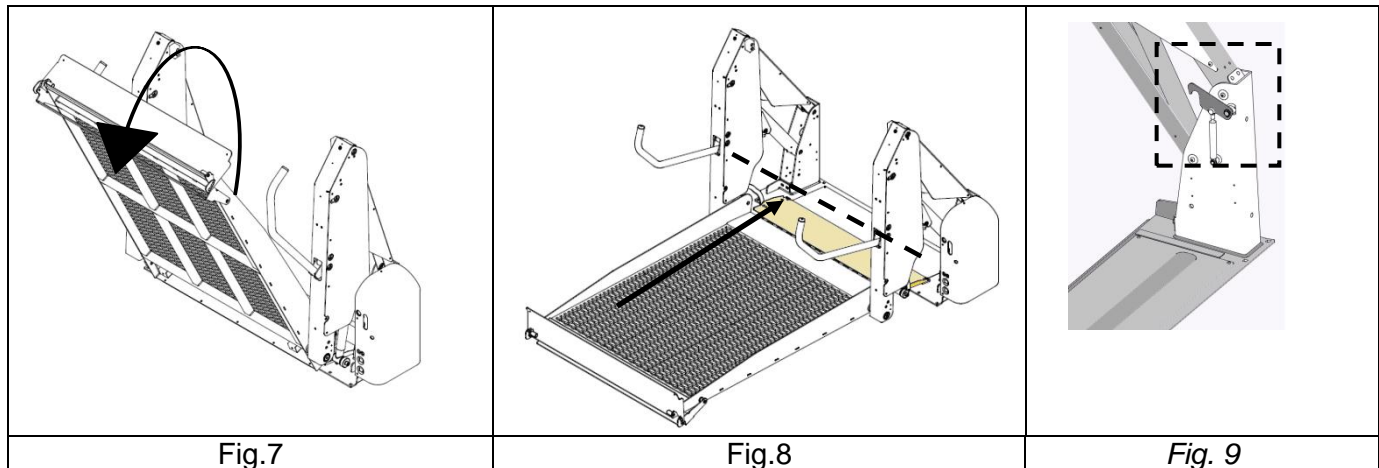
Fig.7

Fig.8

Fig. 9

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#### 6.4. Lowering

#### WARNING

Once disabled person is on the platform or before starting the lifting/lowering operation, the wheelchair **MUST BE** braked.

The descent is a vertical transfer movement in an arc trajectory that brings the loading platform from the floor level of the vehicle up to the ground keeping it horizontal (**Fig.10**).

Follow these steps for the lowering manoeuvre:

- push and keep pushing **lowering** button **3** (see **Fig.2 Chapter 5** until platform;
- Platform reaches the ground and it joins it (**Fig.11**);
- release the button.

Once stopped at the ground level, platform is ready to be loaded or unloaded.

**NOTE** You can accomplish the same task using the wireless remote control (optional) or the joystick command (optional).

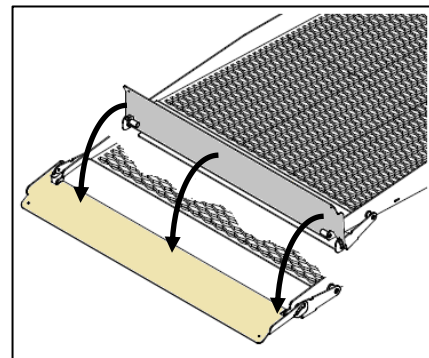
Fig. 11

 **CAUTION** Ensure that the vehicle is parked on a flat surface, horizontal and free of clutter.

 **DANGER** It is mandatory to apply the brakes of the wheelchair before starting the lowering manoeuvre

 **CAUTION** If the platform stops before reaching the ground, during the descent, check for the presence of an obstacle that has induced the tilt, thus causing the intervention of the anti-tilt protection.

 **CAUTION** Make sure that each command corresponds to the relative movement of the platform without jamming and unusual noises.

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### 6.5. Lifting

During the upward movement, the platform always remains flat, approaching the loading platform of the vehicle in an arc trajectory.

Follow these steps for the upward movement:

- Push and keep pushing up button **2** (see **Fig.2 Chapter 5** until platform reaches vehicle deck level (**Fig.12**);
- release the button.

**NOTE** You can accomplish the same task using the wireless remote control (optional) or joystick command (optional).

#### DANGER

The brakes on the wheelchair must necessarily be applied before starting the upward movement.

#### CAUTION

The overloading protection is activated if a load exceeding 125% of the permissible load is present on the platform, blocking the upward movement.

#### CAUTION

Make sure that each command corresponds to the relative movement of the platform without jamming and unusual noises.

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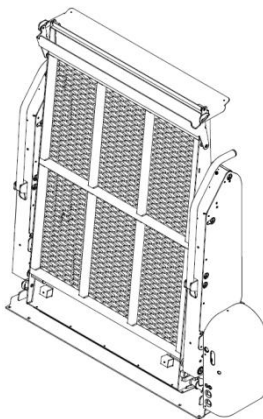
Fig. 13

## 6.6. Stowing

The stowing is a rotational movement that takes platform from the horizontal loading position on the vehicle floor to the vertical closing position.

After making sure that no object has been forgotten on the loading platform, proceed as follows for the closing manoeuvre:

- Push and keep pushing **stowing**, button **4** (see **Fig.2 Chapter Errore. L'origine riferimento non è stata trovata.**) until the platform reaches the closure position by rotating (**Fig.13**);
- release the button only after the pump has stopped identified by the interruption of the relevant noise.

### NOTE

It is possible to carry out the same operation by using the wireless remote control (optional).

### NOTE

With opening/closing doors automatic system (optional), by pushing and keeping pushing **stowing**, button **4** (see **Fig.2 Chapter 5** after platform stowing, vehicle doors (of lift installation area) closes themselves automatically; do not release the button until vehicle doors complete closure.

**WARNING**  
Make sure that the relative movement of the platform corresponds to each command without jamming and unusual noises.

**WARNING**  
If by pushing stowing button **4** for (see **Fig.2 Chapter 5** platform stops and therefore does not close, verify that no objects have remained on the platform and safety pressure switch was activated by them; in this case, deploy the platform, as described in **Chapter 5** remove the objects from the platform and then repeat the stowing manoeuvre.

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### 6.7. Disabling the system

At the end of the operations:

Rotate the key (**Fig. 14**) IN THE SWITCH in anticlockwise direction as far as the warning light, located on the remote-control panel or in vehicle cabin.

- Check that the warning light indicating “platform pulling out” is switched off.
- Remove the key from the switch and keep it into vehicle cabin.
- place both inside the cabin.

### 6.8. Control before vehicle movements

Make sure that passenger compartment warning light is turned OFF when present the communication from wheelchair lift at the vehicle.

#### CAUTION

Before starting the vehicle verify that passenger compartment warning light is turned off and check that the enable key is in OFF position.

#### CAUTION

When using the wireless remote control and doors automatic opening / closing system, do not disable the system before returning to the vehicle.

#### CAUTION

To avoid draining the batteries of the vehicle, remember to disable the system after each use of the lift.

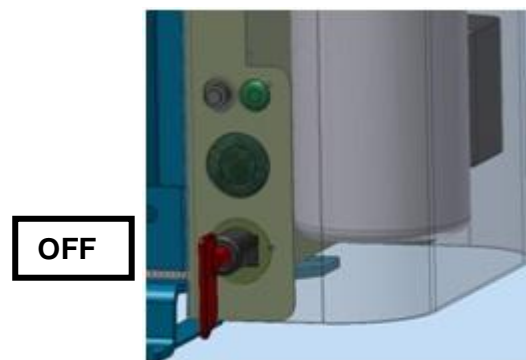
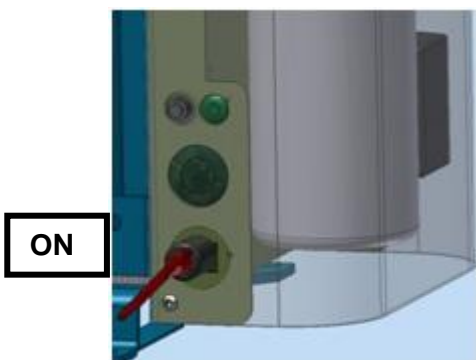
Fig.14

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### 6.9. Expected conditions of special use

The conditions involved are climatic (temperatures extremely low or high) or environmental ones (dusty, muddy, damp areas or areas at high altitude, etc.) forcing the machine to operate in a bearing way.

In any case, it is important to check that:

- electric components keep isolation features unchanged as well as good conditions of cables cleaning and flexibility;
- hydraulic installation components keep fittings features and hydraulic jacks capacity as well as rubber pipelines capacity unchanged;
- mechanical components are always well-protected from rust;
- hydraulic oil level into the tank is always at an optimal level;
- electrolyte level and charge vehicle battery are always at an optimal level;
- hydraulic oil viscosity complies with climate, in order not to have oil too fluid in a warm climates and oil too thick in a cold climates.

### 6.10. Emergency Operation

Emergency manual controls are to be used to completely move the lift in case of lift electric or in case of lack of current supply.

Emergency manual controls must be exclusively used to help the disabled passenger during lifting/lowering onto/ from the vehicle and to close the opened lift, allowing to the vehicle to start again.

We are going to describe a complete cycle of the lift use (exit, lifting, lowering and entry) when it is exclusively moved by means of emergency manual controls.

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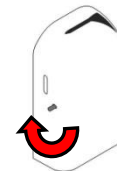
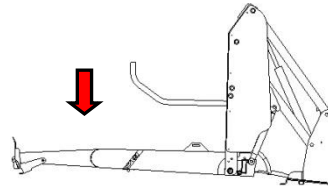
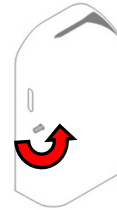
On the hydraulic unit box, turn counterclockwise lowering valve black handle (marked by its relevant label).

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The platform will open and will descend until coming in contact with the ground or until the closure of the lowering valve.

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Before any other operation, close the lowering valve, by turning clockwise its handle.

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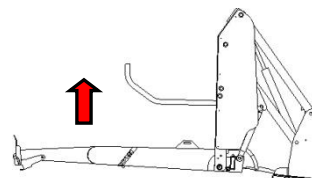
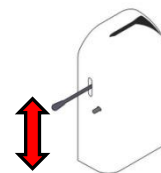
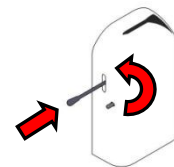
Tighten the lever, in the scope of delivery, in its seat (inside the vertical slot marked by its relevant label).

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Play the manual pump wby means of alternating vertical movements.

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Platform will go up until the manual pump works and the lift will close if you continue driving beyond vehicle deck level.

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## 7. MAINTENANCE

Proper maintenance allows you to work in complete safety and ensures longevity of the wheelchair lift "i-class".

User is therefore obliged to follow the instructions described in the following pages, strictly respecting the relevant deadlines.

### WARNING

User is allowed to carry out only controls listed and the operation listed in *Chapters.7.1* e *7.2*.

### WARNING

In order to ensure functionality and safety of the lift and maintain the warranty valid, user is obliged to carry out routine inspection in an authorized Mobility Networks Holdings Ltd. centre, strictly respecting the relevant deadlines.

### WARNING

In case of anomalies and malfunctions of the lift, user is obliged to carry out special inspection visits in an authorized Mobility Networks Holdings Ltd. centre.

### WARNING

Do not allow unauthorized personnel to carry out interventions on the lift in any manner.

### WARNING

After **2000 HOURS** of work and anyway after **10 YEARS** of service, lift must undergo an inspection and a comprehensive review by an authorized Mobility Networks Holdings Ltd. centre which will issue a new manual with a new assessment of the first commissioning.  
**OTHERWISE, THE LIFT MUST BE REMOVED FROM SERVICE.**

### 7.1 Periodic visual checks

Find following visual checks and the relevant frequencies at which user must perform them in order to ensure the complete efficiency of the lift.

#### BEFORE EACH USE CHECK:

- the integrity of the spiral cable of the wired remote control;
- there are no missing or damaged parts of the machine;
- electrical cables and pipes run freely;
- there are no hardened parts, jams, unusual noises or bad contacts.

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**EVERY WEEK CHECK:**

- vehicle battery is charged;
- control oil level:

In order to ensure to the electro-pump an optimally flow of hydraulic oil, it is necessary that the tank is always full of oil.

It is possible to check the hydraulic oil level into the tank by unscrewing and removing the cap with the platform in all out position and at ground level. If it is necessary to fill up oil **TEXACO RANDO HDZ 15**, put oil into the tank as far as having restored the maximum level (about 2 cm. from the filler pipe union). At the end of the operation, screw the cap.

**WARNING**

When filling up oil, do not mix oils having different features and gradations.

**WARNING**

When checking and filling up oil, keep platform on the ground.

**EVERY MONTH CHECK:**

- there are no oxidation points in the structure;
- there are no loose screws;
- the safety label and stickers are visible and legible;
- lift functions correctly when empty, by performing a complete cycle of deploying, lowering to the ground, ascent and closure.

**WARNING**

If there are anomalies or malfunctions, do not use the lift and contact an authorized Mobility Networks Holdings Ltd. centre.

**WARNING**

Special inspection interventions should be performed only by authorized Mobility Networks Holdings Ltd. centres which will record the action in the appropriate form in Cap.7.4.

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## 7.2 Clening of the loading platform

The loading platform of the lift should be kept clean from dirt, mud and any other deposit in order to preserve the anti-skid characteristics.

### WARNING

For cleaning the platform do not use jets of water at high pressure but degreasing agents.

### WARNING

The oily rags should not be crowded together because in addition to being pollutants, they represent a major risk of fire. Therefore place them in appropriate closed containers and dispose them as required by the ruling law.

### WARNING

The degreasing products should be disposed in accordance with anti-pollution regulations. Do not pour these substances on the ground, in drainage pipes, sewers and waterways.

### WARNING

During the manoeuvres on the platform always use shoes with non-slip soles.

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---

### 7.3 Routine inspections

Compliance with the deadlines of ordinary inspections allows you to work in complete safety and ensures greater longevity to the " i-class", lift, in addition to maintaining the warranty valid until maturity. The user is obliged to annually inspect the lift in an authorized **Mobility Networks Holdings Ltd.** centre, which will perform the checks listed below and any relevant interventions necessary and will compile the related forms attached here below.

#### WARNING

When returning the vehicle verify that the Service Centre has compiled and signed the form attached below, related to the ordinary inspection just executed.

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#### 7.4 Special Inspection

In case you encounter anomalies or failures during the use of the lift or during the periodic visual controls (*Chapter 7.1*), do not use the lift and contact an authorized **Mobility Networks Holdings Ltd.** centre. It will perform the necessary checks and any relevant interventions necessary and will compile the relevant form attached here below.

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**Mobility Networks**

12 Estuary View Business Park, Whitstable, Kent, CT5 3SE, UK  
Tel: +44 (0)1227 505022 Fax: +44 (0)1227 505023 [www.mobilitynetworksgroup.com](http://www.mobilitynetworksgroup.com)

Mobility Networks is a trading name of Mobility Networks (Holdings) Ltd. Registered in England & Wales number 08295602

# M@bility

NETWORKS

## Mobility Networks - European Headquarters

Mobility Networks (Holdings) Ltd  
12 Estuary View Business Park  
Whitstable  
Kent  
CT5 3SE  
United Kingdom

**Tel:** +44(0) 1227 505022

**Fax:** +44(0) 1227 505023

**Email:** [info@mobilitynetworksgroup.com](mailto:info@mobilitynetworksgroup.com)

**Web:** [www.mobilitynetworksgroup.com/en](http://www.mobilitynetworksgroup.com/en)

## Mobility Networks - UK

Passenger Lift Services  
Unit 2 Summit Crescent,  
Smethwick,  
West Midlands  
B66 1BT  
United Kingdom

**Tel:** +44(0) 0121 552 0660

**Fax:** +44(0) 0121 552 0200

**Email:** [enquiries@pls-access.co.uk](mailto:enquiries@pls-access.co.uk)

**Web:** [www.passengerliftservices.co.uk](http://www.passengerliftservices.co.uk)

## Mobility Networks - Italy

Via Matteotti 43  
Gatteo A Mare, FC  
47043  
Italy

**Tel:** +39 0547 179 64 18

**Fax:** +39 0547 195 05 75

**Email:** [leandro@mobilitynetworksgroup.com](mailto:leandro@mobilitynetworksgroup.com)

**Web:** [www.mobilitynetworksgroup.com/it](http://www.mobilitynetworksgroup.com/it)

## CAROL System S.p.a.

Via F.lli Cervi 13  
Palidano di Gonzaga (Mn)  
46023  
Italy

**Tel:** +39 0376 536 766

**Fax:** +39 0376 536 768

**Email:** [caroil@caroil.com](mailto:caroil@caroil.com)

**Web:** [www.caroil.com](http://www.caroil.com)

## Mobility Networks - Nordic ApS

Linde Allé 1  
DK-7330 Brande  
Denmark

**Tel:** +45 4234 3333

**Email:** [ricko@mobilitynetworks.dk](mailto:ricko@mobilitynetworks.dk)

**Web:** [www.mobilitynetworksgroup.com/da](http://www.mobilitynetworksgroup.com/da)

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**Tel:** +40 (0)749 000 100

**Email:** [sales@mobilitynetworks.ro](mailto:sales@mobilitynetworks.ro)

**Web:** [www.mobilitynetworksgroup.com/ro](http://www.mobilitynetworksgroup.com/ro)

## Mobility Networks - Australia & New Zealand

**Email:** [stuart@mobilitynetworksgroup.com](mailto:stuart@mobilitynetworksgroup.com)

**Web:** [www.mobilitynetworksgroup.com/en-gb](http://www.mobilitynetworksgroup.com/en-gb)

## Mobility Networks - North America

1 Delta Park Blvd, Unit 17  
Brampton  
ON L6T 5G1  
Canada

**Tel:** +1 905 672 8338

**Toll free:** +1 844 359 3737

**Email:** [info@mobilitynetworksna.com](mailto:info@mobilitynetworksna.com)

**Web:** [www.mobilitynetworksna.com](http://www.mobilitynetworksna.com)